


# MAXTV

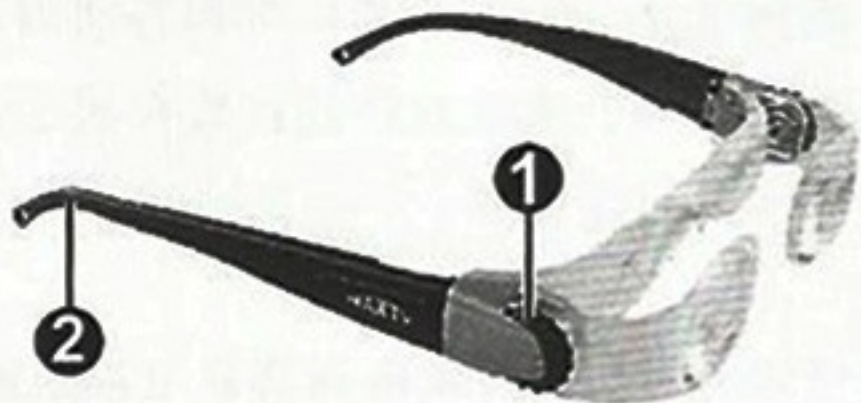

This type of television glasses has novel designs and modern style. It is suitable for nearsighted people (range of vision: 0~-300 degrees) to watch TV. It has such functions as magnifying television images and narrowing television image distance. Furthermore, images are clear and natural without distortion so as to avoid eyestrain.

# Adjustment

The optical characteristics of the MAXTV have been calculated for a distance of 3 m. The MAXTV offers a diopter adjustment (option for correcting defective vision) of 0~-3 dpt. It can be adjusted for each eye individually. You can also use the diopter adjustment to adapt the glasses to other distances besides 3 m.

Position yourself at the desired distance from the television and put on your MAXTV.

- For a better fit the earpieces [2] of the MaxDetail allow themselves to be adjusted.

Keep your left eye closed and turn the knurled ring [1] until the television image seen by your right eye is in sharp focus.

Do the same with the other eye.

## Technical data

Article number	NO.7102-J
Diopter adjustment for each eye individually	0 ~ -3 dpt
Magnification	2.1 x
Field of vision	9° = 1 m / 3 m
Pupil distance	60 mm ... 68 mm
Weight	49 g